sellick®



sellick[®] Cricoid Pressure Trainer

essible affordable functional

а



Proudly distributed by MENTONE in Australia and New Zealand.

www.inovus.org


sellick[®] Cricoid Pressure Trainer

sellick[®] is the cricoid pressure trainer that allows healthcare professionals to learn, practice, retain and reproduce the correct technique for effective cricoid pressures.

sellick[®] gives real time feedback on cricoid pressures allowing training clinicians to repeatedly perform the 'Sellick's Manoeuvre' until they develop muscle memory for the correct pressure required in this procedure. Repeated practise in a simulated environment allows clinicians to perform this technique with confidence when in a clinical environment.

The simulator consists of an anatomically correct 'neck section' with thyroid cartilage, cricoid cartilage and proximal trachea, covered by a simulated skin. The proprietary pressure transfer system, which connects the neck section to the base of the simulator, gives a real to life feel of 'give' as the cricoid is depressed under pressure.

Real time cricoid pressures are displayed on a back lit LED interface. Downwards pressures are displayed in Newtons up to a maximum of 35 Newtons. The simulator has an integrated timer that starts and stops on application of pressure. Cricoid pressures above 35 Newtons are deemed excessive and indicated by a red flashing LED system located around the edge of the neck piece. Lateral pressures are indicated by a blue LED system located at the edge of the neck piece.

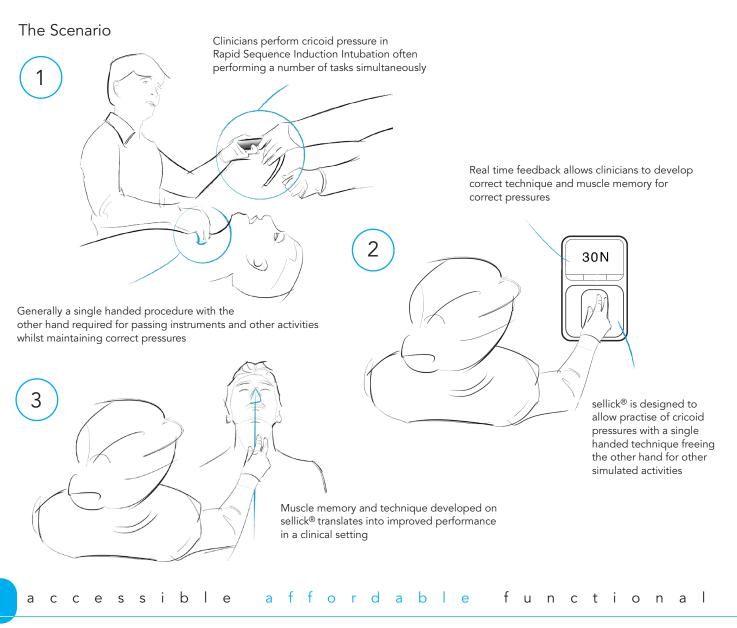

sellick[®] features



For further information on the full range of products by Inovus Medical please contact our distribution partners in Australia and New Zealand:



E: sales@mentone-educational.com.au T: 03 9547 6638 W: www.mentone-educational.com.au

ccessible



а

a f f o r d a b l e



a l

function