

Nasco **Life/form**®

ISO 9001 and
ISO 13485

Injectable Training Model

LF00952U & LF00953U Instruction Card

About the Simulator

Life/form® Injectable Training Model

Nasco developed a lifelike trainer to enhance training for intradermal, subcutaneous, and intramuscular injecting. The material feels like real human flesh. It can be pinched and punctured for injections repeated times!

General Instructions for Use

Dry injecting is recommended. The trainer will reject water or any liquid injected into the trainer. Solvents or corrosive materials will damage the simulator.

Care of the Simulator

Normal soil can be removed with mild soap and water. Nasco Cleaner will remove stubborn stains. Apply Nasco Cleaner to the soiled area and wipe clean with a soft cloth or paper towel. Allow to dry thoroughly before storing. Never place the simulator on newsprint or any kind of printed paper, plastic, or ball-point pen. These materials will transfer indelible stains.

Available Supplies

LF01049U Suture Kit with DVD Collection
LF07000U Suture Replacement
LF07000(A)U Suture Replacement, Pkg. of 5

LF09919U Nasco Cleaner
LF03461U Optional Carry Case
C00879U Mayo Hegar Needle Holder



LF00952U

LF00953U


SN

Note: Save this card should you need to reference the Serial Number above.

Other Available *Life/form*® Simulators

LF00890U Interactive Suture Trainer, White
LF00892U Interactive Suture Trainer, Black
LF00961U Intramuscular Injection Simulator
LF01008U Intradermal Injection Simulator
LF01028U Suture & Stapling Practice Arm
LF01034U Suture & Stapling Practice Leg
LF01042U Suture Kit

LF01046U Facial Suturing Module Set
LF01047U Pediatric Suture Head Kit
LF01131U Venipuncture & Injection Demonstration Arm
LF01184U Venatech IM & SubQ Simulator, White
LF01185U Venatech IM & SubQ Simulator, Black
WA18238U Diabetic Injection Pad
WA28395U Injection Belly

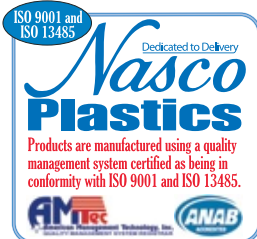

Nasco

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901

eNasco.com
1-800-558-9595

E-mail: lifeform@eNasco.com
COPYRIGHT © NASCO 2013 PRINTED IN U.S.A. NP 062-13