



ENEMA ADMINISTRATION SIMULATOR LF00957U INSTRUCTION MANUAL



ISO 9001 and
ISO 13485

Nasco Plastics
ISO 9001 and ISO 13485

Products are manufactured using a quality management system certified as being in conformity with ISO 9001 and ISO 13485.

Life/form® Products by Nasco

About the Simulator

The **Life/form**® Enema Administration Simulator had been designed to demonstrate and practice enema procedures. Standard apparatus can be used to introduce fluids. This simulator must be treated like a live patient or damage will result.

List of Components

- 1 enema bag
- 1 latex glove
- Nasco Hevi-Lube lubricant
- 18" vinyl tube
- Carrying case


Internal Structure

All aspects of the simulator are life-like in every detail, and the action of the unit is extremely realistic. The upper buttock is flexible so that the trainee must raise the buttock in order to locate and make insertion into the anus. Simulated bowel will accommodate the introduction of fluids.

General Instructions for Use

Set Up Procedure

1. Fill flexible Enema Administrator Unit with clear water.
2. Hang a bag a maximum of 18" above the simulator.

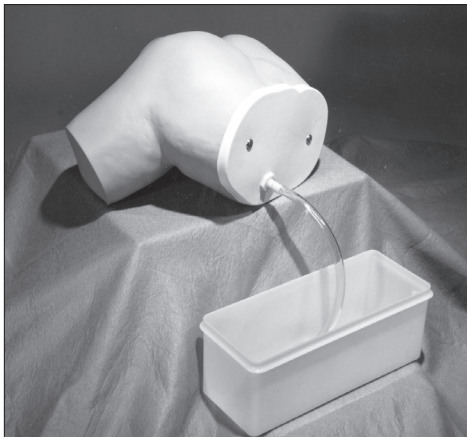

3. Prior to each use of the simulator, the anal orifice and enema catheter **MUST** be lubricated with Nasco Hevi-Lube Lubricant (LF01050U). Do not use substitutions. (CAUTION: Do not use large amounts of Nasco Hevi-Lube. Over application may block drain tube.)
4. The drain tube, which is stored in the case, must be attached to the connector on top of the simulator before the introduction of fluid. Place the drain tube in a sink, pan, or bucket (not included) for continuous drainage of administered fluids.

Procedures That Can Be Performed on This Simulator

The standard enema procedures can be performed using conventional enema apparatus. No special equipment is required.

Care of the Simulator

Normal soil can be removed with mild soap and lukewarm water. Use Nasco Cleaner (LF09919U) to remove stubborn stains. Spray soiled area with cleaner and wipe clean with soft cloth or paper towel.

Before storage, drain completely. Hold simulator upside down and drain all water. Remove 18" drain tubing. Dry all surfaces.

Cautions

Solvents or corrosive materials will damage the simulator. Never place simulator on any kind of printed paper or plastic. These materials will transfer an indelible stain. Ball-point pens will also make a permanent stain.

Supplies/Replacement Parts for Enema Administration Simulator

- LF09919U** Nasco Cleaner
- LF01050U** Nasco Hevi-Lube
- LF01024U** 18" Drain Tubing
- LF01098U** Fluid Administration Set

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (Light)
LF00855U Male Catheterization
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00999U Pediatric Injectable Head
LF01005U First Aid Arm
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01034U Suture Practice Leg
LF01036U Spinal Injection
LF01037U Hemodialysis Practice Arm
LF01038U Episiotomy Suturing Set
LF01042U Suture Kit
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
- LF01087U** Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Infant Intraosseous Infusion
LF01121U Advanced IV Arm
LF01131U Venipuncture and Injection Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01143U Testicular Exam
LF01152U Male & Female Catheter
LF01155U Advanced CPR Dog
LF01162U Venatech IV Trainer
- LF01174U** NG Tube & Trach Skills
LF01184U Venatech IM & Sub Q
LF01193U Special Needs Baby
LF03000U Series
LF03601U Adult Airway Management Trainer with Stand
LF03602U Adult Airway Management Manikin
LF03609U Child Airway Management Trainer with Stand
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03623U Infant Airway Management Trainer with stand
LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer Torso
LF03693U **Basic Buddy**® CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03709U Infant **CRiSis**™ Manikin
LF03720U **Baby Buddy**™ Infant CPR Manikin
LF03750U Bariatric CPR Manikin
LF03770U Chest Tube
LF03953U **CRiSis**™ Manikin, complete
LF03955U Deluxe **CRiSis**™ Manikin
LF03956U Deluxe "Plus" **CRiSis**™ Manikin
LF03965U Adult **CRiSis**™ Auscultation Manikin
LF03966U Adult **CRiSis**™ Auscultation Manikin with ECG Simulator
LF04000U **GERI**™/**KERI**™ Manikin Series
LF04200U Adult Sternal Intraosseous Infusion
LF06001U **CPR Prompt**® Adult/Child Manikin
LF06012U **CPR Prompt**® Infant Manikin
LF06200U **CPR Prompt**® Keychain Rescue Aid
LF06204U **CPR Prompt**® Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901

1.800.558.9595

eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 1999 PRINTED IN U.S.A. NP 157/RV 7-15