

Nasco Dedicated to Delivery **Life/form**®

**Suture and Stapling
Practice Arm
LF01028U
Instruction Manual**



ISO 9001 and
ISO 13485

Nasco Plastics
ISO 9001 and ISO 13485

Products are manufactured using a quality management system certified as being in conformity with ISO 9001 and ISO 13485.

Life/form® Products by Nasco

Life/form®

5
Year
Warranty

About the Simulator

This practice and demonstration suture arm should be considered disposable, but with proper care and treatment, it will serve its purpose for a long period of time.

The three “wounds” provided should be considered only samples of what can be done. With a sharp scalpel, additional “wounds” can be cut at any desired depth or location. Each can be repeatedly sutured until the skin at any one site becomes so heavily perforated that it will no longer support more suturing.

The “wounds” can be bandaged with most standard adhesive bandages. Residual adhesive can be removed with Glue-Off (LF01023U).



Care of Simulator

Do not use providone-iodine prep solutions. These solutions will indelibly stain the arm.

Normal soil accumulated on the surface of the simulator can be removed with mild soap and lukewarm water. Use Nasco Cleaner (LF09919U) to remove stubborn stains from simulator. Simply spray soiled area and wipe clean with a soft cloth or paper towels.

When not in continuous use, it is recommended to store the simulator in the storage box furnished, or cover it with a cloth or sheet of plastic.

Cautions

Solvents or corrosive materials will damage the simulator. Never place simulator on any kind of printed paper or plastic. These materials will transfer indelible stains. Ball-point pens will also make indelible stains.

Supplies/Replacement Parts for Suture Practice Arm


LF01011U Suture Tool Kit

LF09919U Nasco Cleaner

Actual product may vary slightly from photo. Nasco reserves the right to change product color, materials, supplies, or function as needed.

Other Available *Life/form*® Simulators

- LF00890U** Interactive Suture Trainer, Light
- LF00892U** Interactive Suture Trainer, Dark
- LF00929U** Surgical Bandaging
- LF01005U** First Aid Arm
- LF01034U** Suture Practice Leg
- LF01042U** Suture Kit
- LF01046U** Facial Suturing Module Set
- LF01047U** Pediatric Suture Head
- LF01063U** Stump Bandaging, Upper
- LF01064U** Stump Bandaging, Lower

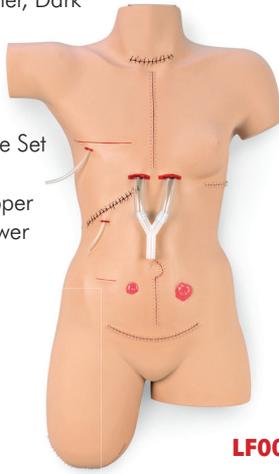

Front



Back

LF00929U

LF01034U



LF01046U

LF00890U



LF01047U



Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1.800.558.9595

eNasco.com • E-mail: lifeform@eNasco.com