



Child Airway Management Trainer with Stand LF03609U Instruction Manual



ISO 9001 and
ISO 13485

Nasco Plastics
ISO 9001 and ISO 13485

Products are manufactured using a quality management system
certified as being in conformity with ISO 9001 and ISO 13485.

Life/form® Products by Nasco

About the Simulator

The **Life/form**® Child Airway Management Trainer is the most realistic simulator available for the training of intubation skills. Nasco has taken great care to create an airway management trainer that is anatomically correct in respect to both size and detail.

Landmarks include nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true vocal cords, cricoid ring, tracheal rings, trachea, esophagus, lungs, and stomach.

The **Life/form**® Child Airway Management Trainer allows you to practice oral, digital, and nasal intubation. Suction techniques can also be performed and evaluated. The simulator was designed to use an uncuffed endotracheal tube measuring up to 5.0 mm I.D.

List of Components

- Child Airway Management Trainer Head with Lungs and Stomach
- Mounting Board
- 8-oz. Pump Spray Lubricant
- Case



Figure 1

Lubrication

Before using the **Life/form**® Child Airway Management Trainer, lubricate both the tube and airway with the Nasco lubricant provided. (**See figure 1.**)

Note: Nasco recommends the use of the provided lubricant or a similar vegetable-based lubricant for the Child Airway Management Trainer. The use of a silicone-based or similar lubricant may cause damage to the simulator, thus voiding Nasco's warranty on the trainer.

Cleaning the Face

Normal soil can be removed from the face of the trainer with mild soapy water. Nasco Cleaner (LF09919U) will remove stubborn stains. Simply apply Nasco Cleaner to the soiled area and wipe clean with a soft cloth or paper towel.

Never place the trainer on any kind of printed paper or plastic. These materials will transfer indelible stains. Ball-point pens will also make indelible stains. Do not use any cosmetics.

Removing the Child Airway Management Trainer Head from the Mounting Board

If you wish to remove the head from the board:

1. Remove the lungs and stomach from the trainer by loosening the clamps that hold them in place. The clamps are loosened by twisting each side of the clamp in opposite directions.
2. Now turn the head until the keys on the neck align with the slots in the mounting bracket and slide out. (**See figure 2.**)

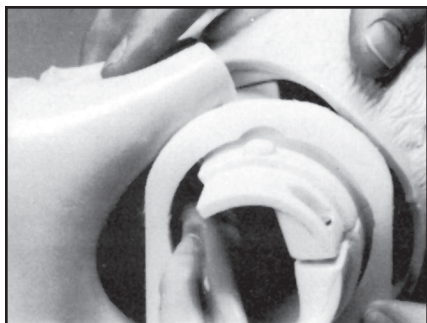


Figure 2

3. To replace the head on the mounting board, simply reverse the previous two steps.

Important — Storage of Your Simulator

It is vital that the packing foam is retained for correct storage of your simulator. The unit must be stored as you received it to avoid damaging the simulator. Follow these steps when placing the simulator in the case for storage or transport.

1. Place the board in the case so that the wide end of the board is positioned under the two cleats inside the case.
2. Making sure the head is in the mounting bracket, turn the head 90° to its left so that it is facing the hinged side of the case.
3. Place the foam block into position over the neck, just below the chin, so that the round hole is up.
4. The case can now be closed. Your trainer will be protected from excessive movement within the case.

Available Supplies

- LF03644U** Pump Stray Lubricant
- LF03627U** Replacement Lungs Set
- LF03628U** Replacement Stomach
- LF03631U** Replacement Headskin with Airway

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (Light)
LF00855U Male Catheterization
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00999U Pediatric Injectable Head
LF01005U First Aid Arm
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01034U Suture Practice Leg
LF01036U Spinal Injection
LF01037U Hemodialysis Practice Arm
LF01038U Episiotomy Suturing Set
LF01042U Suture Kit
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Infant Intraosseous Infusion
LF01121U Advanced IV Arm
LF01131U Venipuncture and Injection Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01143U Testicular Exam
LF01152U Male & Female Catheter
LF01155U Advanced CPR Dog
LF01162U Venatech IV Trainer
LF01174U NG Tube & Trach Skills
LF01184U Venatech IM & Sub Q
LF01193U Special Needs Baby
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer with Stand
LF03602U Adult Airway Management Manikin
LF03609U Child Airway Management Trainer with Stand
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03623U Infant Airway Management Trainer with Stand
LF03632U Child Intraosseous Infusion/Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer Torso
LF03693U **Basic Buddy**® CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03709U Infant **CRiSis**™ Manikin
LF03720U **Baby Buddy**™ Infant CPR Manikin
LF03750U Bariatric CPR Manikin
LF03770U Chest Tube
LF03953U **CRiSis**™ Manikin, Complete
LF03955U Deluxe **CRiSis**™ Manikin
LF03956U Deluxe "Plus" **CRiSis**™ Manikin
LF03965U Adult **CRiSis**™ Auscultation Manikin
LF03966U Adult **CRiSis**™ Auscultation Manikin with ECG Simulator
LF04000U **GERI**™/**KERI**™ Manikin Series
LF04200U Adult Sternal Intraosseous Infusion
LF06001U CPR Prompt® Adult/Child Manikin
LF06012U CPR Prompt® Infant Manikin
LF06200U CPR Prompt® Keychain Rescue Aid
LF06204U CPR Prompt® Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1.800.558.9595

eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 1998 PRINTED IN U.S.A. NP 28-96/RV 6-15