

STEPS TO CONNECT TO THE

RD-953




HEALTH PLANET APP INSTRUCTIONS




1

OPEN THE TANITA HEALTH PLANET APP

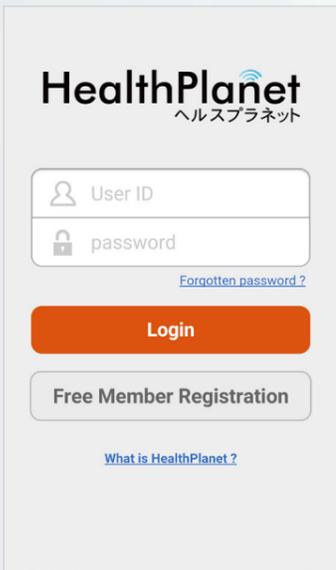


2

SELECT "INPUT DATA FROM BCM"

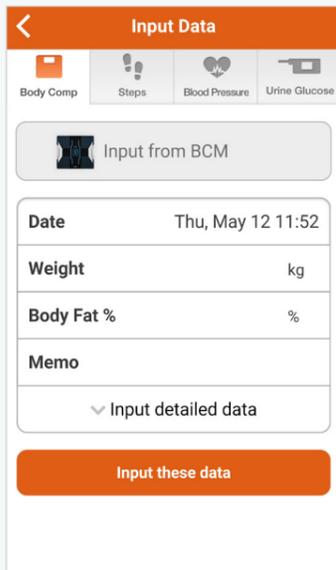


3

SELECT MODEL RD-953

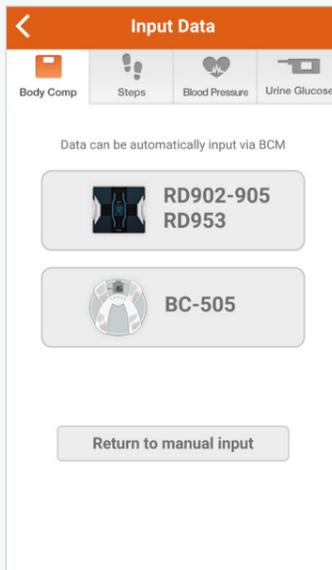


4

SELECT "ADD DEVICE"

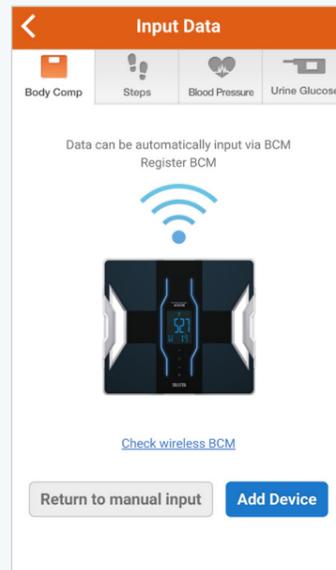


5

ENSURE BCM IS OFF AND HOLD DOWN THE 0. SELECT "OK" WHEN INDICATED

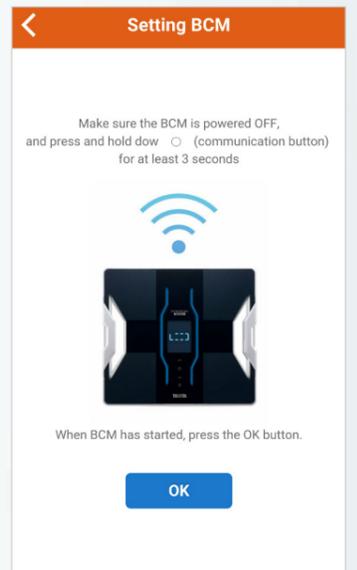


6

CHOOSE A USER NUMBER AND SELECT "REGISTRATION"

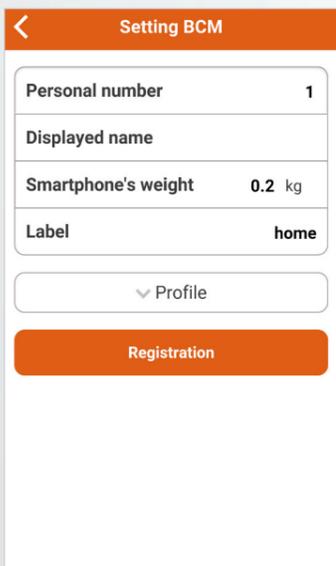


7

SELECT "OK" TO OVERWRITE PREVIOUS USER DATA

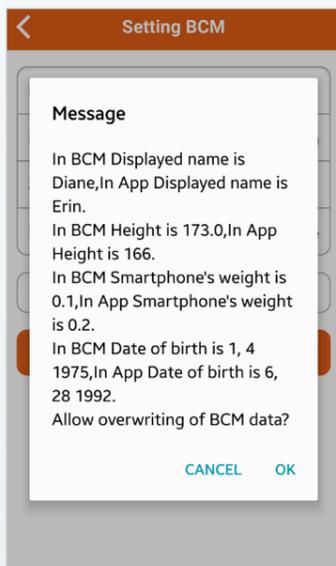


8

WHEN INDICATED, STEP ONTO THE SCALE

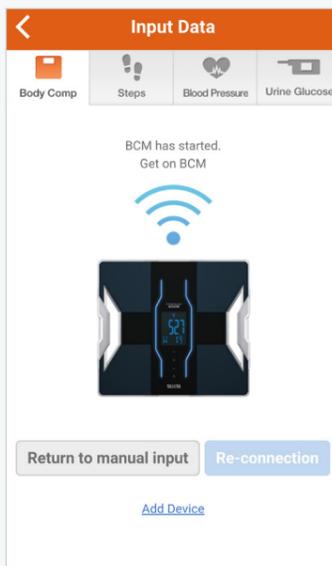

9

WHEN THE READINGS APPEAR ON YOUR DEVICE STEP OFF. SELECT "COMPLETED" TO SAVE

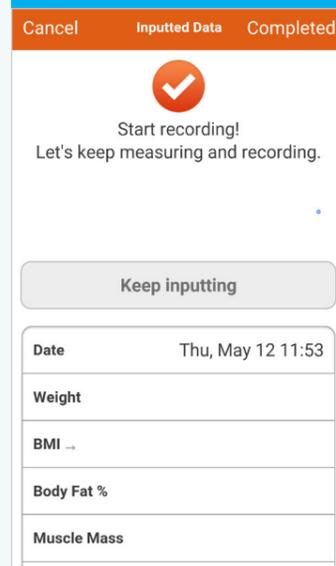

YOU ARE NOW CONNECTED!

KNOW YOUR BODY: Download our FREE Healthy Range Chart

